



BUFFERED SALINE

**Rinsing and Storage Solution
for All Soft Contact lenses
Unit Volume: 15 ml**

Do not use as disinfection solution.

Contents:

A sterile, preservative-free, buffered isotonic aqueous solution containing Sodium Chloride 0.57% , Potassium Dihydrogen Phosphate, Sodium Hydroxide and Purified Water ad q.s.

- Ampoules are sealed to ensure maximum protection of solution.
- Packed in single-dose disposable ampoules.
Highly hygienic and convenient.
- As a rinsing and storage solution for all soft contact lenses after cleaning, disinfecting and before wearing.
- Preservative free, suitable for sensitive eyes.
- Isotonic and buffered for maximum comfort.

To use:

1. Detach an ampoule from remaining block.
2. Twist off the top.
3. Use the solution at once.

Explanatory Symbols:



European (EU/EC) Authorised Representative



Sterilized by Filtration



Expiry date



Lot number



Warning

Precautions:

1. Wash Hand thoroughly before use.
2. Do not mix with other fluids except as directed.
3. Do not use if allergic to any of the declared ingredients in the product.
4. Do not use concurrently with any eye medicament except on medical advice.
5. Once opened, discard any unused solution immediately.
6. Do not touch the ampoule tip with other surface.
7. Do not use if seal is broken.
8. Do not use as disinfection solution.
9. If you experience any eye discomfort, stop using immediately and consult eye practitioner.
10. Protect from direct sunlight.
11. Keep out of reach of children.
12. Store at room temperature (15°C-30°C).

13. Not for injection.
14. Solution is clear, do not use if the solution is turbid.
15. Do not use if eye is lacerated or object is embedded.



Manufacturer & Brand Owner

Opto-Pharm Pte Ltd
13 Tuas Avenue 12 Singapore
639035

www.optopharm.com.sg