



BUFFERED SALINE

Buffered Saline Solution is used as a rinsing solution and storage solution, for all contact lenses. It is also used as a storage medium in the thermal case for heat disinfecting.

Buffered Saline Solution does not have any therapeutic and or diagnostic indications and claims.

Content:

A buffered, sterile and isotonic solution containing Boric Acid and Edetate Disodium.

Intended use:

Buffered Saline Solution is used as a rinsing solution and storage solution for all soft contact lenses, after cleaning, disinfecting and before wearing. It is also used as a storage medium in the thermal case for heat disinfecting.

Indications (Uses):

Buffered Saline Solution is indicated for use in daily rinsing and storing of soft contact lenses.

Intended User:

It is compatible with all soft contact lenses. The product may be used with daily or disposable lenses or lenses prescribed for frequent replacement, Silicone Hydrogel lenses and *coloured lenses.

Directions:

To open for the first time:

Unscrew the cap and discard the safety ring

After removing the safety ring, screw in the cap tightly until the cone pin punctures the bottle tip and breaks the seal.

The cone pin will also reseal the opened bottle.

Replace cap tightly after use.

Step by Step direction:

After cleaning and disinfection, rinse the lens thoroughly with Buffered Saline Solution for 5 seconds on each side either on the palm or using the forefinger of the other hand or between the thumb and forefinger until debris is removed. Rinse the thermal case thoroughly with solution first before filling thermal case with solution for heat disinfection.

Action:

Rinse off disinfecting solution before contact lens wear.

Contraindications (Reasons not to use):

Do not use if allergic to any of the declared ingredients in the product.

Warning:

Eye problems with contact lenses and lens care products could result in corneal infection and/or ulcers and loss of vision.

It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Failure to discard solution from lens case after each use or discard any unused solution after 90 days or use of water to care for contact lenses may lead to contamination resulting in eye injury and potential loss of vision.

If irritation, redness or change of vision occurs, stop usage and consult an eye care practitioner.

Do not use with any heat disinfecting methods.

Discard the solution in the lens after use, wash the lens case, clean, and air dry, including the covers. Never use the same lens case for more than 3 months.

Do not use water to rinse the contact lens.

Do not use if seal is broken.

**Coloured lenses for cosmetic purposes are non-medical uses ascribed to this product and are not within the scope of CE certification. Please be aware that product performance and/or safety has not been evaluated for those purposes*

Precautions:

Do not mix with other fluids except as directed.

Do not touch bottle tip to the eye or any surface.

Once opened, discard any unused solution after 30 days.

Protect from direct sunlight.

Store at room temperature (15 °C - 30 °C).

Replace cap tightly after use.

Keep out of reach of children.



European (EU/EC) Authorised Representative

S.B. Pharma GmbH Bunsenstr.
14, 53121 Bonn, Germany
Tel: (+49) 228-52266916
Fax: (+49) 228-52266916

Manufacturer:

Opto-Pharm Pte Ltd
13 Tuas Avenue 12
Singapore 639035
www.optopharm.com.sg
Tel: 65-68614801
Fax: 65-68614806



Unit Volume: 500ml