



ONE STEP SOLUTION

One Step Solution is used as a cleaning, disinfecting, storing, rinsing, lubricating and comfort enhancing solution for all soft and hard contact lenses.

One Step Solution does not have any therapeutic and or diagnostic indications and claims.

Longer lasting comfort
Extends wearing time
Lubricating the lens
Moisturizing and wetting
Suitable for all types of contact lenses

Content:

A Phosphate buffered, sterile and isotonic solution containing Poloxamer 407, EDTA, Polyhexamethylene biguanide (PHMB), Propylene glycol and other ingredients

Intended use:

One Step Solution is used as a cleaning, disinfecting, storing, rinsing, lubricating and comfort enhancing solution for all soft and hard contact lenses.
One Step Solution does not have any therapeutic and or diagnostic indications and claims.

Indications (Uses):

One Step Solution is indicated for use in daily cleaning, disinfecting, and storing of soft and hard contact lenses under non-heating environments.

Intended User:

It is compatible with all types of contact lenses. The product may be used with daily or extended wear lenses, disposable lenses or lenses prescribed for frequent replacement, Silicone Hydrogel lenses and *coloured lenses.

Directions:

To open for the first time:

1. Unscrew the cap and discard the safety ring.
2. After removing the safety ring, screw in the cap tightly until the cone pin punctures the bottle tip and breaks the seal.
3. The cone pin will also reseal the opened bottle.
4. Replace cap tightly after use.

Step by step direction:

Step 1: To clean

Place a few drops of One Step Solution on the lens. Rub gently for 20 seconds. Rinse the lens with more solution to remove debris.

Step 2: Storage

Place the lenses in the lens case which is filled with One Step Solution. Soak for at least 6 hours for lens lubrication and disinfection.

Step 3: Rinse

Rinse soaked lenses with One Step Solution before placing into eyes.

Repeat procedure with the other lens. One Step Solution may be used to rinse the storage case.

Action:

Helps prevent the formation of irritating deposits on the lens surface. Kills harmful microorganisms on the lens while your lenses are soaking. One Step Solution forms a lubricating layer on the lenses by retaining moisture, giving the eyes long lasting comfort.

Contraindications (Reasons not to use):

Do not use if allergic to any of the declared ingredients in the product.

Warning:

Eye problems with contact lenses and lens care products could result in corneal infection and/or ulcers and loss of vision. It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case. Failure to discard solution from lens case after each use or discard any unused solution after 60 days or use of water to care for contact lenses may lead to contamination resulting in eye injury and potential loss of vision.

If irritation, redness or change of vision occurs, stop usage and consult an eye care practitioner.

Do not use with any heat disinfecting methods.

Discard the solution in the lens after use, wash the lens case, clean, and air dry, including the covers. Never use the same lens case for more than 3 months.

Do not use water to rinse the contact lens. Do not use if seal is broken.

**Coloured lenses for cosmetic purposes are non-medical uses ascribed to this product and are not within the scope of CE certification. Please be aware that product performance and/or safety has not been evaluated for those purposes*

Precautions:

Do not mix with other fluids except as directed.

Do not touch bottle tip to the eye or any surface.

Once opened, discard any unused solution after 60 days.

Protect from direct sunlight.

Store at room temperature (15°C - 30°C).

Replace cap tightly after use.

Keep out of reach of children.



European (EU/EC) Authorised Representative

S.B. Pharma GmbH Bunsenstr.
14, 53121 Bonn, Germany
Tel: (+49) 228-52266916
Fax: (+49) 228-52266916

Manufacturer:

Opto-Pharm Pte Ltd
13 Tuas Avenue 12
Singapore 639035
www.optopharm.com.sg
Tel: 65-68614801
Fax: 65-68614806



Unit Volume: 60ml, 120ml, 240ml, 260ml, 350ml, 360ml, 500ml