



## **SALINE SOLUTION FOR EYES AND WOUND IRRIGATION**

**Saline solution for eyes and wound irrigation is a phosphate buffered, sterile and isotonic solution containing Sodium Chloride, Potassium Dihydrogen Phosphate, Sodium Hydroxide and Purified Water. It also acts as an irrigation solution for eye and wound wash.**

### **Content:**

A phosphate buffered, sterile and isotonic solution containing Sodium Chloride, Potassium Dihydrogen Phosphate, Sodium Hydroxide and Purified Water.

### **Intended use:**

Saline solution for eyes and wound irrigation acts as an irrigation solution for eyes and wound wash. It also aids in the removal of foreign materials from the eyes or wound.

### **Indications (Uses):**

Saline solution for eyes and wound irrigation is indicated for use as emergency eyes and wound wash and flushing/irrigating the eyes and affected areas to remove loose foreign materials, air pollutants, chlorinated water and to refresh irritated or tired eyes.

### **Intended User:**

Intended for affected individuals (no age and gender restriction).

### **Directions:**

1. Detach an ampoule and twist off the top.
2. Flush the affected eyes or areas as needed.
3. Each ampoule is good for single use only.

### **Action:**

Saline solution for eyes and wound irrigation flushes and irrigates the eyes and affected areas by removing loose foreign materials, air pollutants, and chlorinated water. It refreshes irritated or tired eyes.

### **Contraindications (Reasons not to use):**

Do not use if allergic to any of the declared ingredients in the product.

### **Warning:**

External use only.

Not for injection.

If eyes pain, irritation, redness condition worsens or persists, discontinue use and consult a doctor.

If eye vision changed or cloudy occurs, eye is lacerated or object is embedded, stop usage and consult a doctor.

### **Precautions:**

Do not mix with other fluids except as directed.

Do not use concurrently with any eye medication except on medical advice.

Do not use with any heat disinfecting method.

Do not touch the ampoule tip to the eye or with any other surface.

Do not use it if allergic to any of the declared ingredients in the product.

Once opened, discard any unused solution immediately.

Do not use if solution changed colour.

Protect from direct sunlight.

Keep out of reach of children.

Store at room temperature (15°C- 30°C).

Do not freeze.



European (EU/EC) Authorised Representative

S.B. Pharma GmbH Bunsenstr.  
14, 53121 Bonn, Germany  
Tel: (+49) 228-52266916  
Fax: (+49) 228-52266916

### **Manufacturer:**

Opto-Pharm Pte Ltd  
13 Tuas Avenue 12  
Singapore 639035  
[www.optopharm.com.sg](http://www.optopharm.com.sg)  
Tel: 65-68614801  
Fax: 65-68614806



**Unit Volume: 5ml, 10ml, 15mL**