



COMFORT DROPS

Comfort Drops is used to rewet, condition, lubricate, clean, and enhance comfort of all types of contact lenses.

- Longer lasting comfort
- Extends wearing time
- Lubricating the lens
- Moisturizing and wetting
- Suitable for all types of contact lenses

Content:

Comfort Drops is a borate buffered, sterile and isotonic solution containing Polyhexamethylene biguanide (PHMB), Edetate Disodium Dihydrate (EDTA), Hydroxypropyl Methylcellulose (HPMC), Poloxamer 407 and other ingredients.

Intended Use:

Comfort Drops is used as a cleaning, biofilm (protein and lipid) removing, wetting, lubricating, and comfort enhancing solution for all soft (including silicone hydrogel lens) and hard contact lenses.

Indications (Uses):

Comfort Drops is indicated for use to lubricate and rewet soft and hard contact lenses during wear. It is compatible with all types of hard lenses and soft contact lenses including silicone hydrogel lenses and *coloured lenses.

Intended User:

It is compatible with all types of contact lenses. The product may be used with daily or extended wear lenses, disposable lenses or lenses prescribed for frequent replacement, silicone hydrogel lenses and *coloured lenses.

Directions:

To open for the first time:

To open for the first time, twist cap in clockwise direction to break cap from the safety ring.

Remove the ring.

Puncture the tip by screwing the cap in clockwise direction until the cone pin punctures the bottle tip and break the seal. The cone pin will also reseal the opened bottle.

Use 1 or 2 drops on contact lens, up to 4 times a day as needed. Replace cap after use.

Once opened, discard any unused solution after 60 days.

Action:

Comfort Drops is used to rewet and lubricate while the lenses are being worn. Follow the step-by-step directions.

Contraindications (Reasons not to use):

Do not use if allergic to any of the declared ingredients in the product.

Warning:

Eye problems with contact lenses and lens care products could result in corneal infection and/or ulcers and loss of vision.

It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Failure to discard solution from lens case after each use or discard any unused solution after 60 days or use of water to care for contact lenses may lead to contamination resulting in eye injury and potential loss of vision.

If you experience eye discomfort, excessive tearing, vision changes or redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner.

Do not use as disinfection solution.

Do not use water to rinse the contact lens.

Do not use with other eye medication except on medical advice.

Do not use if sterile packaging/ seal is damaged/ broken.

**Coloured lenses for cosmetic purposes are non-medical uses ascribed to this product and are not within the scope of CE certification. Please be aware that product performance and/or safety has not been evaluated for those purposes.*

Precautions:

Do not mix with other fluids except as directed.


Do not touch bottle tip to the eye or any surface. Replace cap after use.

Once opened, discard any unused solution after 60 days.

Protect from direct sunlight.

Store at room temperature (15 °C - 30 °C).

Keep out of reach of children.

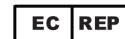

EU Authorised Representative:

S.B. Pharma GmbH
Bunsenstr.14,
53121 Bonn, Germany
Tel: (+49) 228-52266916
Fax: (+49) 228-52266916

Manufacturer:

Opto-Pharm Pte Ltd
13 Tuas Avenue 12 Singapore 639035

www.optopharm.com.sg

Tel: 65-68614801

Fax: 65-68614806



Unit Volume: 10mL