

MULTI-PURPOSE SOLUTION WITH HA

Multi-Purpose with HA is a borate buffered, sterile, and isotonic solution containing Polyhexanide (PHMB), Edetate Disodium Dihydrate (EDTA), Hydroxypropyl Methylcellulose (HPMC), Poloxamer 407, Sodium Hyaluronate, Propylene Glycol, and other ingredients.

Cleans and disinfects
Longer lasting comfort
Extends wearing time
Lubricating the lens
Moisturizing and wetting
Suitable for all hard and soft contact lenses

Content:

A borate buffered, sterile and isotonic solution containing Polyhexanide (PHMB), Edetate Disodium Dihydrate (EDTA), Hydroxypropyl Methylcellulose (HPMC), Poloxamer 407, Sodium Hyaluronate, Propylene Glycol, and other ingredients.

Intended Use:

Multi-Purpose Solution with HA is used as a cleaning, biofilm (protein and lipid) removing, disinfecting, storing, rinsing, wetting, lubricating, moisture retaining and comfort enhancing solution for all hard and soft contact lenses, including *coloured lenses and silicone hydrogel lenses.

Indications (Uses):

Multi-Purpose Solution with HA is indicated for use as a cleaning, biofilm (protein/lipid) removing, disinfecting, storing, rinsing, wetting, lubricating, moisture retaining and comfort enhancing solution of all hard and soft contact lenses.

Intended User:

Intended for individuals who wear hard and soft contact lenses. It is compatible with all hard and soft contact lenses. The product may be used with daily disposable lenses or lenses prescribed for frequent replacement, Silicone Hydrogel lenses and *coloured lenses.

Directions:

To open for the first time:

Detach an ampoule and twist off the top.
Each ampoule is good for single use only.

Step by Step direction:

Step 1: To Clean

Place a few drops of Multi-Purpose Solution with HA on the lens. Rub gently for 20 seconds. Rinse the lens with more solution to remove debris.

Step 2: Storage

Place the lenses in the lens case which is filled with Multi-Purpose Solution with HA. Soak for at least 4 hours for lens lubrication and disinfection.

Step 3: Rinse

Rinse soaked lenses with Multi-Purpose Solution with HA before placing into eyes. Repeat procedure with the other lens.

Multi-Purpose Solution with HA may be used to rinse the storage case.

Action:

Loosens and removes accumulations of film, debris and deposits from contact lenses, through mechanical washing. It kills harmful microorganisms on the lens while your lenses are soaking. Multi-Purpose Solution with HA forms a lubricating layer on the lenses by retaining moisture, giving the eyes long lasting comfort.

Contraindications (Reasons not to use):

Do not use if allergic to any of the declared ingredients in the product.

Warning:

Eye problems with contact lenses and lens care products could result in corneal infection and/or ulcers and loss of vision.

It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Failure to discard solution from lens case after each use or discard any unused solution immediately or use of water to care for contact lenses may lead to contamination resulting in eye injury and potential loss of vision.

If irritation, redness or change of vision occurs, stop usage and consult an eye care practitioner.

Do not use with any heat disinfecting methods.

Discard the solution in the lens after use, wash the lens case, clean, and air dry, including the covers.

Never use the same lens case for more than 3 months.

Do not use with other eye medication except on medical advice.

Do not use water to rinse the contact lens.

Do not use if sterile packaging/ seal is damaged/ broken.

**Coloured lenses for cosmetic purposes are non-medical uses ascribed to this product and are not within the scope of CE certification. Please be aware that product performance and/or safety has not been evaluated for those purposes.*

Precautions:

Do not mix with other fluids except as directed.

Do not touch ampoule tip to the eye or any surface.

Do not freeze.

Once opened, discard any unused solution immediately.

Protect from direct sunlight.

Store at room temperature (15 °C - 30 °C).

Keep out of reach of children.



EU Authorised Representative:

S.B. Pharma GmbH
Bunsenstr.14,
53121 Bonn, Germany
Tel: (+49) 228-52266916
Fax: (+49) 228-52266916

Manufacturer:

Opto-Pharm Pte Ltd
13 Tuas Avenue 12 Singapore 639035
www.optopharm.com.sg
Tel: 65-68614801
Fax: 65-68614806



Unit Volume: 10mL, 15mL