

## MULTI-PURPOSE SOLUTION WITH HA

**Multi-Purpose with HA is a borate buffered, sterile, and isotonic solution containing Polyhexanide (PHMB), Edetate Disodium Dihydrate (EDTA), Hydroxypropyl Methylcellulose (HPMC), Poloxamer 407, Sodium Hyaluronate, Propylene Glycol, and other ingredients.**

Cleans and disinfects  
Longer lasting comfort  
Extends wearing time  
Lubricating the lens  
Moisturizing and wetting  
Suitable for all hard and soft contact lenses

### Content:

A borate buffered, sterile and isotonic solution containing Polyhexanide (PHMB), Edetate Disodium Dihydrate (EDTA), Hydroxypropyl Methylcellulose (HPMC), Poloxamer 407, Sodium Hyaluronate, Propylene Glycol, and other ingredients.

### Intended Use:

Multi-Purpose Solution with HA is used as a cleaning, biofilm (protein and lipid) removing, disinfecting, storing, rinsing, wetting, lubricating, moisture retaining and comfort enhancing solution for all hard and soft contact lenses, including \*coloured lenses and silicone hydrogel lenses.

### Indications (Uses):

Multi-Purpose Solution with HA is indicated for use as a cleaning, biofilm (protein/lipid) removing, disinfecting, storing, rinsing, wetting, lubricating, moisture retaining and comfort enhancing solution of all hard and soft contact lenses.

### Intended User:

Intended for individuals who wear hard and soft contact lenses. It is compatible with all hard and soft contact lenses. The product may be used with daily disposable lenses or lenses prescribed for frequent replacement, Silicone Hydrogel lenses and \*coloured lenses.

### Directions:

#### To open for the first time:

Unscrew the cap and discard the safety ring. After removing the safety ring, screw in the cap tightly until the cone pin punctures the bottle tip and breaks the seal. The cone pin will also reseal the opened bottle.  
Replace cap tightly after use.

#### Step by Step direction:

##### Step 1: To Clean

Place a few drops of Multi-Purpose Solution with HA on the lens. Rub gently for 20 seconds. Rinse the lens with more solution to remove debris.

##### Step 2: Storage

Place the lenses in the lens case which is filled with Multi-Purpose Solution with HA. Soak for at least 4 hours for lens lubrication and disinfection.

##### Step 3: Rinse

Rinse soaked lenses with Multi-Purpose Solution with HA before placing into eyes. Repeat procedure with the other lens.

Multi-Purpose Solution with HA may be used to rinse the storage case.

### Action:

Loosens and removes accumulations of film, debris and deposits from contact lenses, through mechanical washing. It kills harmful microorganisms on the lens while your lenses are soaking. Multi-Purpose Solution with HA forms a lubricating layer on the lenses by retaining moisture, giving the eyes long lasting comfort.

### Contraindications (Reasons not to use):

Do not use if allergic to any of the declared ingredients in the product.

### Warning:

Eye problems with contact lenses and lens care products could result in corneal infection and/or ulcers and loss of vision.

It is essential that you follow your eye care practitioner's directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Failure to discard solution from lens case after each use or discard any unused solution after 90 days or use of water to care for contact lenses may lead to contamination resulting in eye injury and potential loss of vision.

If irritation, redness or change of vision occurs, stop usage and consult an eye care practitioner.

Do not use with any heat disinfecting methods.

Discard the solution in the lens after use, wash the lens case, clean, and air dry, including the covers.

Never use the same lens case for more than 3 months.

Do not use with other eye medication except on medical advice.

Do not use water to rinse the contact lens.

Do not use if sterile packaging/ seal is damaged/ broken.

*\*Coloured lenses for cosmetic purposes are non-medical uses ascribed to this product and are not within the scope of CE certification. Please be aware that product performance and/or safety has not been evaluated for those purposes.*

### Precautions:

Do not mix with other fluids except as directed.

Do not touch bottle tip to the eye or any surface.

Do not freeze.

Replace cap tightly after use.

Once opened, discard any unused solution after 90 days.

Protect from direct sunlight.

Store at room temperature (15 °C - 30 °C).

Keep out of reach of children.



### EU Authorised Representative:

S.B. Pharma GmbH  
Bunsenstr.14,  
53121 Bonn, Germany  
Tel: (+49) 228-52266916  
Fax: (+49) 228-52266916

### Manufacturer:

Opto-Pharm Pte Ltd  
13 Tuas Avenue 12 Singapore 639035  
[www.optopharm.com.sg](http://www.optopharm.com.sg)  
Tel: 65-68614801  
Fax: 65-68614806



**Unit Volume: 100mL, 120mL, 240mL, 260mL, 350mL, 360mL, 500mL**