

MULTI-PURPOSE SOLUTION

Multi-Purpose Solution is used as a cleaning, biofilm (protein and lipid) removing, disinfecting, storing, rinsing, wetting, lubricating and comfort enhancing solution for all contact lenses, including coloured and silicone hydrogel lenses.

Longer lasting comfort
Extends wearing time
Lubricating the lens
Moisturizing and wetting
Suitable for soft contact lenses

Content:

A borate buffered, sterile and isotonic solution containing Hydroxypropyl Methylcellulose (HPMC), Poloxamer 407, Edetate Disodium Dihydrate (EDTA), Polyhexamethylene biguanide (PHMB).

OP MPS contains HPMC, which forms a lubricating layer on your lenses by retaining moisture, giving your eyes long lasting comfort.

The high ionic molecules of OP MPS pull charged protein molecules away from the lenses, thus enhancing the release of protein deposits from the lens.

Intended Use:

Multi-Purpose Solution is used as a cleaning, biofilm (protein and lipid) removing, disinfecting, storing, rinsing, wetting, lubricating, and comfort enhancing solution for all soft contact lenses, including *coloured lenses and silicone hydrogel lenses.

Indications (Uses):

Multi-Purpose Solution is indicated for use in daily cleaning, rinsing, disinfecting (non-heat), and storage of soft contact lenses, including coloured lenses and silicone hydrogel lenses, as recommended by your eye care practitioner.

Intended User:

It is compatible with soft contact lenses. The product may be used with daily or extended wear lenses, disposable lenses or lenses prescribed for frequent replacement, *coloured lenses and silicone hydrogel lenses.

Directions:

To open for the first time:

1. Unscrew the cap and discard the safety ring.
2. After removing the safety ring, screw in the cap tightly until the cone pin punctures the bottle tip and breaks the seal. The cone pin will also reseal the opened bottle.

Step by Step direction:

Step 1: To clean and remove protein

Place a few drops of Multi-Purpose Solution on the lens. Rub gently for 20 seconds. Rinse the lens with more solution to remove debris.

Step 2: Storage

Place the lenses in the lens case which is filled with Multi-Purpose Solution. Soak for at least 4 hours for lens lubrication and moisturizing (For extra comfort), protein removal, and disinfection.

Step 3: Rinse

Rinse soaked lenses with Multi-Purpose Solution before placing into eyes. Repeat procedure with the other lens.

Multi-Purpose Solution may be used to rinse the storage case.

Note: For “No Rub”, go straight to Step 2 if you are using disposable lenses, 30 days or less. “No Rub” feature is subjected to your eye practitioner’s instruction.

Action:

Helps prevent the formation of irritating deposits on the lens surface. Kills harmful microorganisms on the lens while your lenses are soaking. Multi-Purpose Solution forms a lubricating layer on the lenses by retaining moisture, giving the eyes long lasting comfort.

Contraindications (Reasons not to use):

Do not use if allergic to any of the declared ingredients in the product.

Warning:

Eye problems with contact lenses and lens care products could result in corneal infection and/or ulcers and loss of vision.

It is essential that you follow your eye care practitioner’s directions and all labeling instructions for proper use of lenses and lens care products, including the lens case.

Failure to discard solution from lens case after each use or discard any unused solution after 90 days or use of water to care for contact lenses may lead to contamination resulting in eye injury and potential loss of vision.

If irritation, redness or change of vision occurs, stop usage and consult an eye care practitioner.

Do not use with any heat disinfecting methods.

Discard the solution in the lens after use, wash the lens case, clean, and air dry, including the covers.

Never use the same lens case for more than 3 months.

Do not use with other eye medication except on medical advice.

Do not use water to rinse the contact lens.

Do not use if sterile packaging/ seal is damaged/ broken.

**Coloured lenses for cosmetic purposes are non-medical uses ascribed to this product and are not within the scope of CE certification. Please be aware that product performance and/or safety has not been evaluated for those purposes.*

Precautions:

Do not mix with other fluids except as directed.

Do not touch bottle tip to the eye or any surface.

Do not freeze.

Replace cap tightly after use.

Once opened, discard any unused solution after 90 days.

Protect from direct sunlight.

Store at room temperature (15 °C - 30 °C).

Keep out of reach of children.

Manufacturer:

Opto-Pharm Pte Ltd
13 Tuas Avenue 12 Singapore 639035
www.optopharm.com.sg
Tel: 65-68614801
Fax: 65-68614806



Unit Volume: 60mL, 100mL, 120mL, 240mL, 260mL, 350mL, 360mL, 500mL